INTRODUCTIE

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HOW TO USE

**ON**

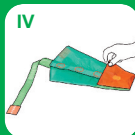
1. Place the Magnide 2in1 across the foot (ensure that the orange flap near the toes is well folded back, so that the magnet makes contact)
2. Pull the stocking over the Magnide 2in1
3. Slide the stocking over the foot, ensuring that the heel of the stocking sits in the right place
4. Make sure that the light green loop sticks out above the stocking so it can be grasped
5. Pull the light green loop to remove the Magnide 2in1 from the stocking

**OFF**

1. Slide Magnide 2in1 over the stocking
2. Fasten the velcro strap firmly just above the ankle
3. Fold the spare material down over the strap as far as possible
4. Fold the stocking over the Magnide 2in1 as far as possible
5. With small movements, starting at the heel, slide the stocking and Magnide 2in1 over the heel

Preparation for re-use

- I Fix the top of the loop to the top of the Magnide 2in1 by using the magnet
- II Turn over the Magnide 2in1 and place it on a flat surface
- III Fold both sides with magnets to the middle, so that the magnets make contact. The orange stitching is now visible on the outside
- IV Close the orange flap (near the toes) over the Magnide 2in1 so that the magnets make contact

SIZING

Art. nr.	Size	Shoe size	
E01540	Medium	≤ 40	
E01541	Large	≥ 41 t/m 45	
E01542	Extra large	≥ 46	
			XL also suitable for extra wide feet with cA upto 28 cm

INTRODUCTION

Magnide 2in1 provides an effective solution for putting on compression stockings.

Your new Magnide 2in1 is characterised by a high level of effectiveness, comfort and quality. As you know, putting on compression stockings is an intensive activity which often causes problems such as damaged stockings or strained muscles. The Magnide 2in1 has been designed to address these problems.

Magnide 2in1 is a valuable aid not just for the stocking wearer but also for their professional carers.

To be sure you get the most out of the Magnide 2in1 read the enclosed product information carefully. If you have any questions, please contact your supplier.

PRODUCT BENEFITS / CHARACTERISTICS

- Reduction of physical strain for both caregiver and stocking wearer
- Increase in independence of the stocking wearer
- Increase in therapy compliance
- Stockings retain therapeutic compression longer
- The Magnide 2in1 is made of innovative, smooth and light material

Use a separate Magnide 2in1 for each patient. If the Magnide 2in1 needs to be used on more than one patient, you must cleanse the product with a 70% ethanolic solution.

WASHING INSTRUCTIONS

- Magnide 2in1 may only be washed by hand, at a low temperature (30°C)
- Dry the product with a towel - never iron
- Do not dry clean or treat with bleach
- Do not expose the product to sources of heat like bright sunlight or heaters

IMPORTANT

- Warning! Do not step on the Magnide 2in1. The highly slippery material makes it a slipping hazard.
- Warning! The Magnide 2in1 contains several magnets. For this reason, keep this product away from pacemakers and defibrillator. The magnetic field may interfere with the proper operation of this equipment (minimum distance = 30 cm).
- When using the Magnide 2in1 it is important to use the correct size for you. Using an incorrect size will prevent the product from working properly, and it may become damaged or wear out too quickly. The correct size is determined by your shoe size.
- Wear special rubber gloves when using the Magnide 2in1. This gives you maximum grip on the stocking, and reduces the physical strength needed.
- This product is guaranteed for a period of six months from the date of purchase against defects resulting from faulty material or construction. Within the guarantee period, the product will be exchanged free of charge in the case of manufacturing and/or material defects. Any exchange under the guarantee will only be carried out if it can be demonstrated by means of a purchase receipt that the date on which the complaint was lodged falls within the guarantee period. To use this service you need to contact your Magnide 2in1 supplier. You are asked to return the product washed and to follow the washing instructions.

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