ADVANCIS

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Utylizacja

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ADVANCIS
WOUND PROBEWound Size and Shape
Assesment Probe

Instructions for use

	Catalogue Number		Consult instructions for use
	Do not re-use		Keep dry
	Sterilized using irradiation		Keep away from sunlight
	Do not use if package is damaged		Manufacturer
	Caution		Authorized representative in the European Community
	Quantity		Batch code
	Unique Device Identifier (UDI carrier including the AIDC and human readable information)		Medical Device
	Date of manufacture/ Country of manufacture		CE Mark for Northern Ireland, EC & EU
	The product has made a financial contribution towards the recovery and recycling of packaging in Europe		Importer
	Do not re-sterilize		Single sterile barrier system
	Use-by date		Indicates the authorised representative in Switzerland

Wound Probe®

Wound size and shape assesment probe

Product description

Advancis Medical Wound Probe® is a disposable wound assessment stick designed to determine the depth of a wound.

Indications

Pressure ulcers, sinuses, wounds with an unknown depth.

Contra-indications

Do not use if wound opening is smaller than width of probe.

Directions for use

Use the probe to assess the wound by inserting into the wound.

If the wound opening is smaller than the width of the probe a different form of assessment should be used and the probe should not be inserted.

Frequency of change

Single use, dispose of after assessment of wound.

Warnings and Precautions

Wound Probe® is a single use product. If used on more than one patient, cross-contamination or infection may result. Opening the wound probe pack compromises the sterile barrier. Any opened, unused wound probes should be discarded.

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