EN Prefilled Syringe with Sterile Water

Prefilled Syringe with Sterile Water with 10% Glycerin

Prefilled Syringe with Sterile Water with 0.9% Sodium Chloride - Saline

Steriflush® prefilled syringes containing Water, 10% Glycerin solution or 0.9% Sodium Chloride solution are intended to inflate the balloon of Foley catheters or wash non-damaged patient skin for medical procedures. Steriflush® luer-slip syringes are filled with clear, colourless and non-pyrogenic solutions. Steriflush® prefilled syringes are sterile disposable syringes made of clear polypropylene with graduation to ensure precise application.

Caution

- Do not use if the package has been opened or damaged.
- Do not use if syringe tip cap or stopper is damaged in any way that suggests or indicates syringe leakage.
- Do not use for any kind of reconstitution of drugs or local I.V. administration. Not for injection.
- Do not use if the solution is not transparent and translucent.
- For single-use only, discard after use.

Instructions for Use

Steriflush® prefilled syringes with Water, 10% Glycerin solution or 0.9% Sodium Chloride solution must be used in line with your hospital's Foley catheter inflation policy and also catheter device manufacturer's guidelines.

1. Using aseptic technique, remove the syringe from the package. Discard if packaging is not intact.
2. Check that syringe tip cap is in place. Inspect for clarity of solution.
3. Remove the tip cap from the sterile syringe ensuring that there is no contamination of the syringe connection.
4. After insertion of the catheter into the bladder, pull the prefilled syringe into the inflation port (see fig. 1). DO NOT inflate the balloon for any reason before inserting the catheter.
5. Push the plunger slowly to make the balloon inflate. In case of extreme plunger resistance, it is recommended that excessive force is not exerted.
6. Twist off the syringe to remove it.
7. Pull back gently on the catheter to make sure the balloon is inflated. At this time you can hook the catheter to the drainage bag.
8. After use, dispose syringe in accordance with recognised procedure in your institution.

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 Reference number	 Batch number	 Medical device	 Unique device identifier	 Manufacturer		
 Date of manufacture	 Expiration date	 Protect from adverse weather conditions	 Keep away from sunlight	 Do not use if package is damaged		
 Do not re-use	 Consult instructions for use	 Caution	 Single sterile barrier system	 Single sterile barrier system with protective packaging outside		
 Medical device has been subjected to ONE of the below sterilization method (please check the sterilization method used in the device packaging label).						
 Sterilized using irradiation	 Sterilized using irradiation fluid path	 Sterilized using ethylene oxide				
 Sterilush Medical Products Zona Industrial 1 - Lote 11 e 14 4450-166 Guilfê, Portugal Portugal - www.sterilush.com					 0344	 PART OF STI GROUP sti Steris International